Ref. 302100 / 302101

DEUTSCH

## Behandlungsstand A6000

Cage de contention A6000 / Cattle Crush A6000 / Behandelbox A6000/ Puesto de tratamiento A6000 /  
Stanza di trattamento A6000

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Additionally required tools for assembly:

- Wrench set
- Hex wrench
- Ladder

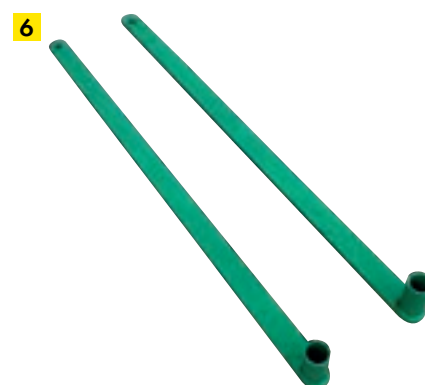
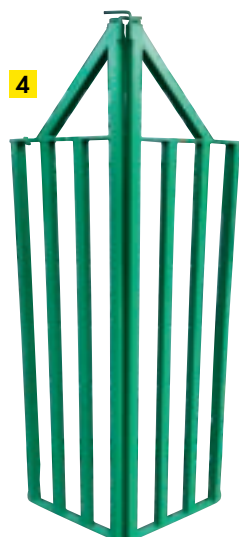
Mounting with two persons in approx. 1 h

# Installation and operating instructions

## Included in delivery:


|          | Qty | Description                                                                        |
|----------|-----|------------------------------------------------------------------------------------|
| <b>1</b> | 1x  | Cattle Crush A6000 green painted (Ref. 302100) or hot-dip galvanised (Ref. 302101) |
| <b>2</b> | 1x  | Rump bar                                                                           |
| <b>3</b> | 1x  | Operating lever for headbail unit                                                  |
| <b>4</b> | 1x  | Baulk gate, double-leaf                                                            |
| <b>5</b> | 1x  | Operating lever for baulk gate                                                     |
| <b>6</b> | 2x  | Flat bar                                                                           |

## Accessories (optional):
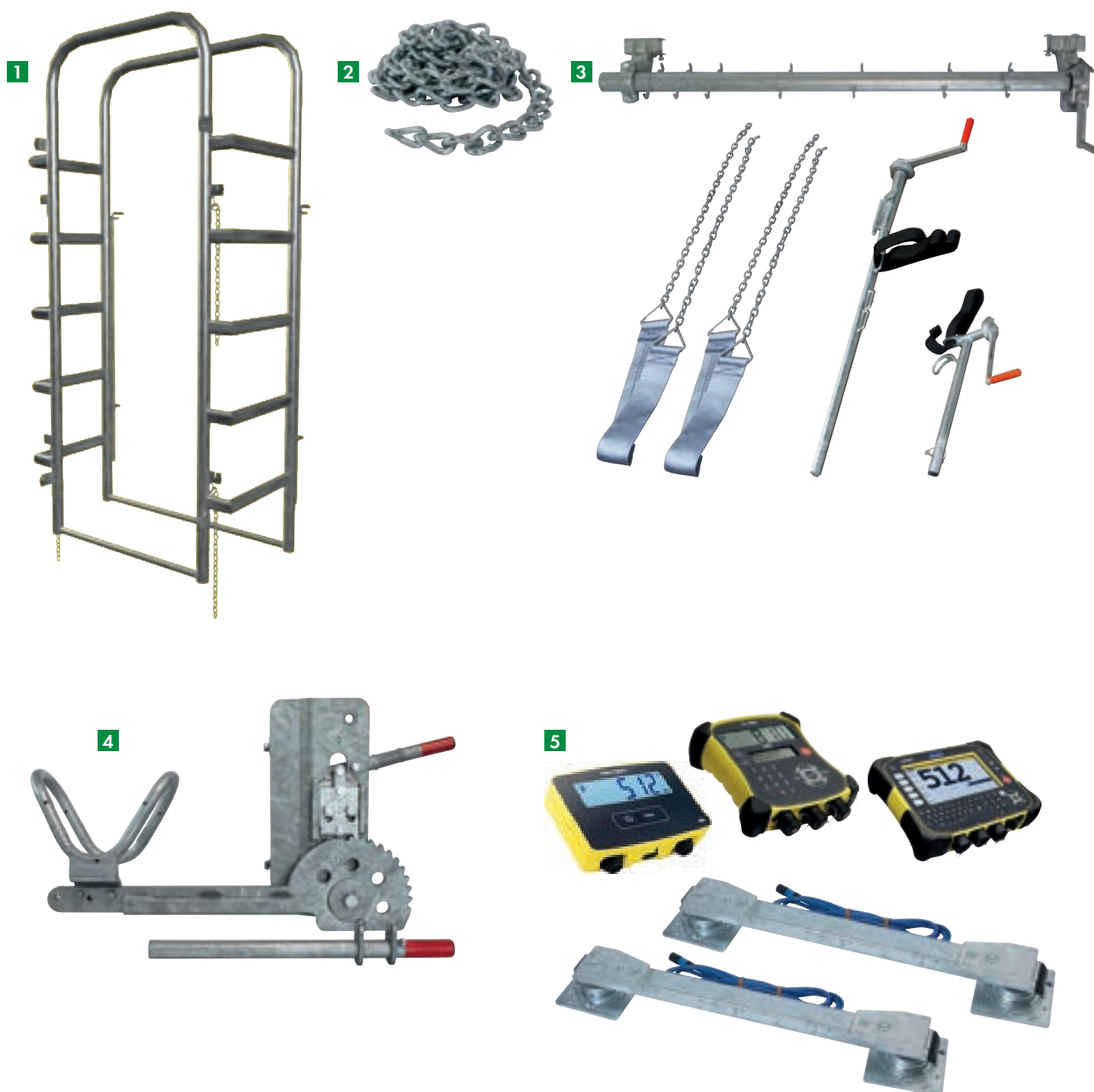
|   | Qty | Description                                                                                                       |
|---|-----|-------------------------------------------------------------------------------------------------------------------|
| 1 | 1x  | Connecting Cage (Ref. 310104)                                                                                     |
| 2 | 1x  | Head-chain-fixation (Ref. 302359)                                                                                 |
| 3 | 1x  | Hoof Care Kit (Ref. 302360)                                                                                       |
| 4 | 1x  | Head Fixation (Ref. 302408)                                                                                       |
| 5 | 1x  | Weighing system:<br>Indicator unit (e.g. Ref. 400067, 400047, 480022, 480023 or 480024) + Load Bars (Ref. 400500) |

## Installation and operating instructions

- The unit is largely pre-assembled.
- All operating elements can be mounted on both sides. The required mounting material is included.
- Before each use, lubricate the sliding rod of the brake with penetrating oil.

- **Quiet working makes it easier to work with animals.**  
Try to guide or entice the animals rather than to drive, push or force them.
- **Wear protective clothing such as gloves, safety shoes with steel toe caps and, when using an angle grinder, suitable safety goggles.**

- 1.** Transport safely via pallet fork.  
Place it on level ground at the end of the working alley.

- .....
- 2.** For efficient and safe working: complete handling system with holding pen (see PATURA main catalogue under „Mobile Handling Systems“).  
Use Connecting Cage **1** and Treatment Cage (Ref. 310103) for safe treatment and as a passage from working alley to the cattle crush.

### 3. Installation + Operation Baulk Gate, double-leaf

- A** Mount the two wings of the baulk gate **4** to the cattle crush **1**.  
 Attach the operating lever for the baulk gate **5** to the frame of the baulk gate **4**.  
 Insert the pipe sleeves of the flat bars **6** into the wings of the baulk gate **4** and bolt the other end of the flat bars to the operating lever **5**.

- B** Close baulk gate **4** before guiding the animal in.

## 4. Installation + Operation Headbail Unit

### A Installation Headbail Unit

Operation from the **left** (animal walking direction):  
Mount the adapter piece with the lever so that the lever is positioned at approx. **5 o'clock**. Ensure that the screw threads face outwards.

Operation from the **right** (animal walking direction):  
Mount the adapter piece with the lever so that the lever is positioned at approx. **7 o'clock**. Ensure that the screw threads face outwards.

**B** Open the headbail unit using operating lever **3** only as wide as necessary for the animal's head to pass through and guide the animal in.

When guiding the animal in, one person must hold the operating lever **3** of the headbail unit in the open position. The baulk gate is closed.

## 5. Catching and treating

**A** Catch the animal with the fully adjustable headbail unit and close it so that the animal cannot move backwards inside the cattle crush. Close the sliding gate.

**B** For safe fixation of the animal:  
Insert rump bar **2** into the guide, push it forward as far as possible and secure it using the angle steel.

**C** Treatments at the head (ear tags, nose ring etc.):  
Open baulk gate **4**: push operating lever **5** forward

Options for fixing the head:

1. With head fixation **4**: see step **7**.

2. With head-chain-fixation **2**:

Hook the chain into lug **A** and lead it over the mouth. Pull it through ring **B**, then upwards over the animal's head to lug **C**, tighten and hook into lug **C**.

## Installation and operating instructions

**D** Treatments at other parts of the animal: open the three side doors on each side.

## E Releasing the animal:

1. Remove rump bar **2** by lifting the angle steel.
2. Keep the rear sliding gate closed.
3. Open baulk gate **4**: push operating lever **5** forward.
4. Open the headbail unit using operating lever **3** and hold it open until the animal has left the crush.

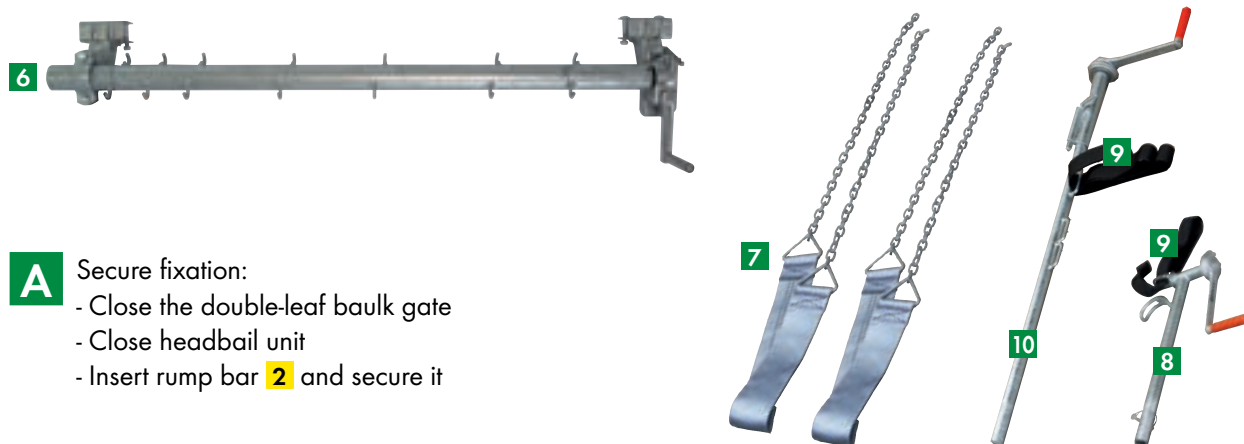
- Remove the crush from the pasture after use. The animals may damage the stand when they access it from outside.
- Clean the crush thoroughly after use and store it in a dry place.
- To ensure proper function and long service life, adequate and regular lubrication of the designated grease nipples is essential.
- The brakes of the headbail unit may only be lubricated with penetrating oil.
- If not used for an extended period, wrap the sliding rod of the brake with an oily cloth.

## 6. Installation + Operation Hoof Care Kit 3 (optional)

### Included in delivery:

|    | Qty | Description       |
|----|-----|-------------------|
| 6  | 1x  | Belly strap winch |
| 7  | 2x  | Belly strap       |
| 8  | 1x  | Front foot winch  |
| 9  | 2x  | Foot strap        |
| 10 | 1x  | Hind foot winch   |

- A** Secure fixation:
- Close the double-leaf baulk gate
  - Close headbail unit
  - Insert rump bar 2 and secure it

- B**
- Hook the belly straps 7 into the frame on the fixed side
  - Lead the straps under the animal through to the other side
  - Hang into the hooks of the belly strap winch 6 and tighten the belly straps slightly
  - The hooks for the belly strap can be moved to adjust to the animal size

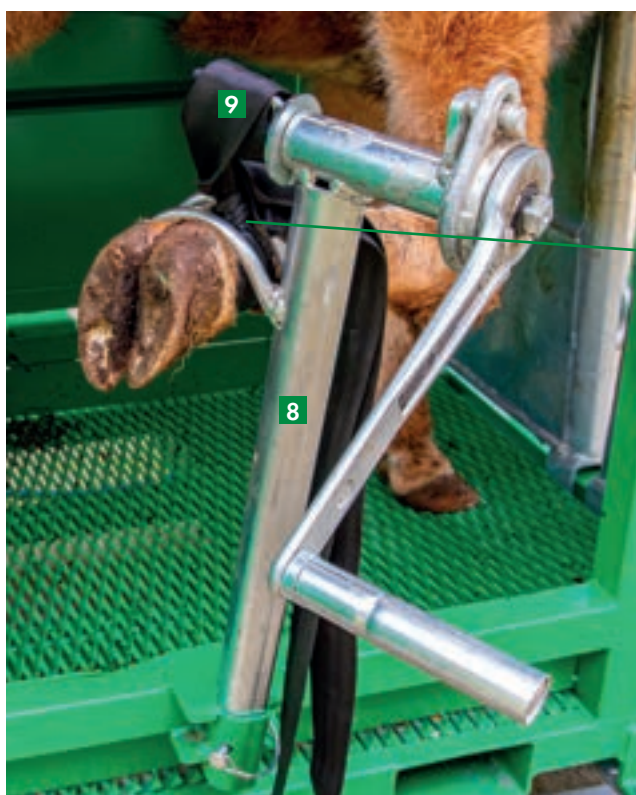
Do not lift the animal too much, otherwise it may become unsafe!



- C**
- Insert the front foot winch **8** into the holder and secure it with a linch pin
  - Wrap foot strap **9** around the front foot as a loop (between dew claw and claw)
  - Guide foot strap **9** through the inside of the foot support and put it into the front foot winch **8**
  - Pull the foot upwards by turning the front foot winch **8** until the foot is firmly supported in the front foot rest

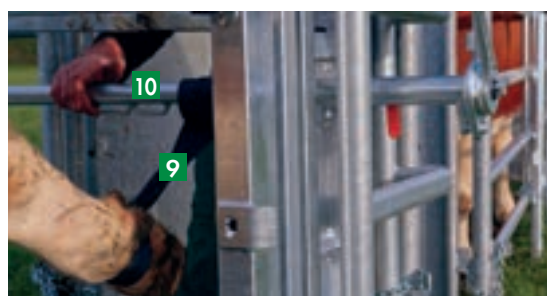- D**
- Insert the hind foot winch **10** through the holes suitable for the animal size above the rear side door and secure it
  - Wrap foot strap **9** around the rear foot (between dew claw and claw) and insert into hind foot winch **10**
  - Pull the foot upwards by turning the hind foot winch **10** until it is firmly against the bar

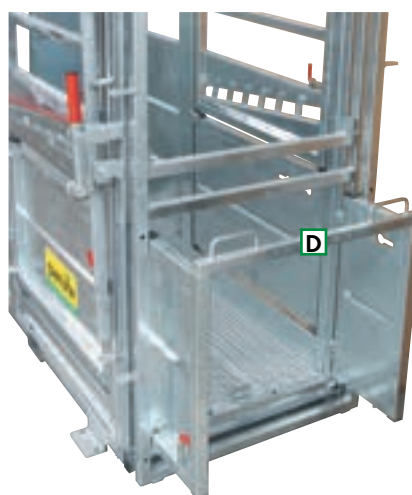- E** If the hind foot winch **10** is used in the rearmost position, bar **D** must be used to lock the folding doors.


## Installation and operating instructions


### 7. Installation + Operation Head Fixation 4 (optional)

#### Included in delivery:
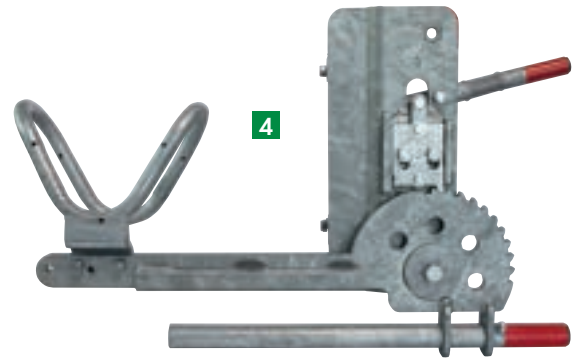
|   | Qty | Description                 |
|---|-----|-----------------------------|
| 4 | 1x  | Head Fixation (Ref. 302408) |

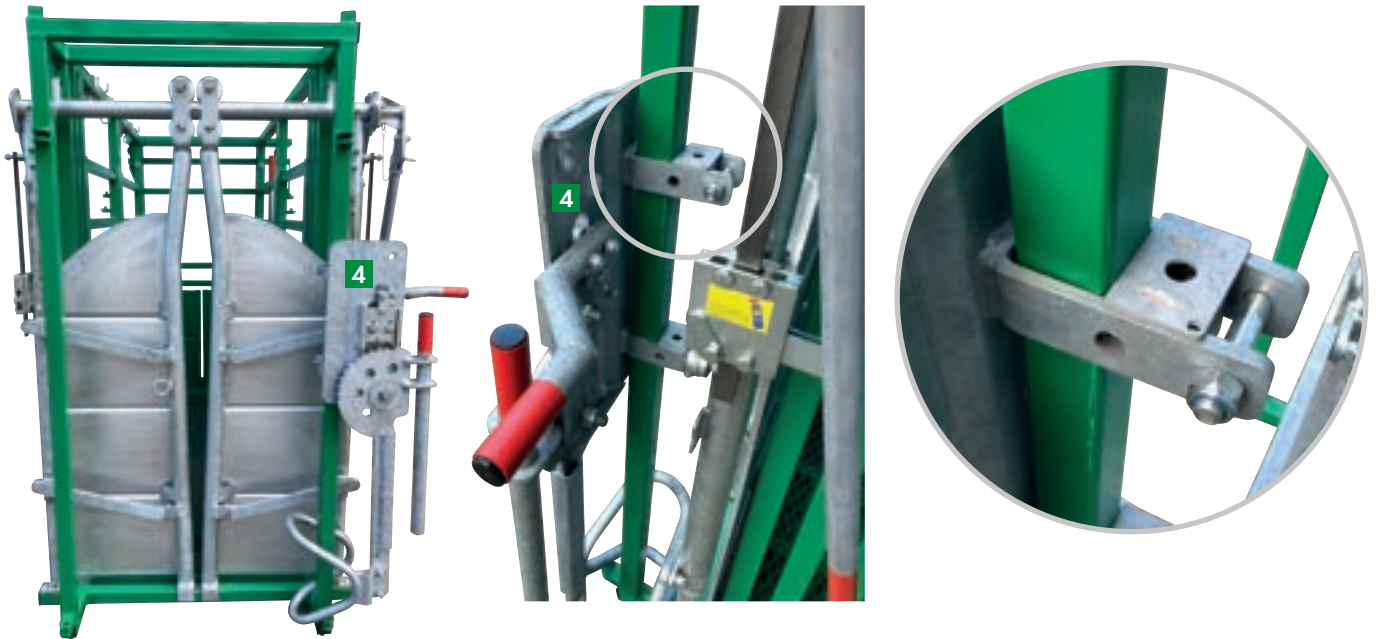- A** Remove the wing of the baulk gate on the operating side (standard: left). Bolt head fixation 4 to the frame using the U-profiles. Mount the head bracket in the outer hole pattern. Then re-install the baulk gate wing.



- B** After the animal has been fixed in the headbail unit, pull the operating lever of the head fixation **E** upwards from the rest position, then press it down to bring the head fixation from below up to the head of the animal.

**C** To release, press the release lever **F** downwards.

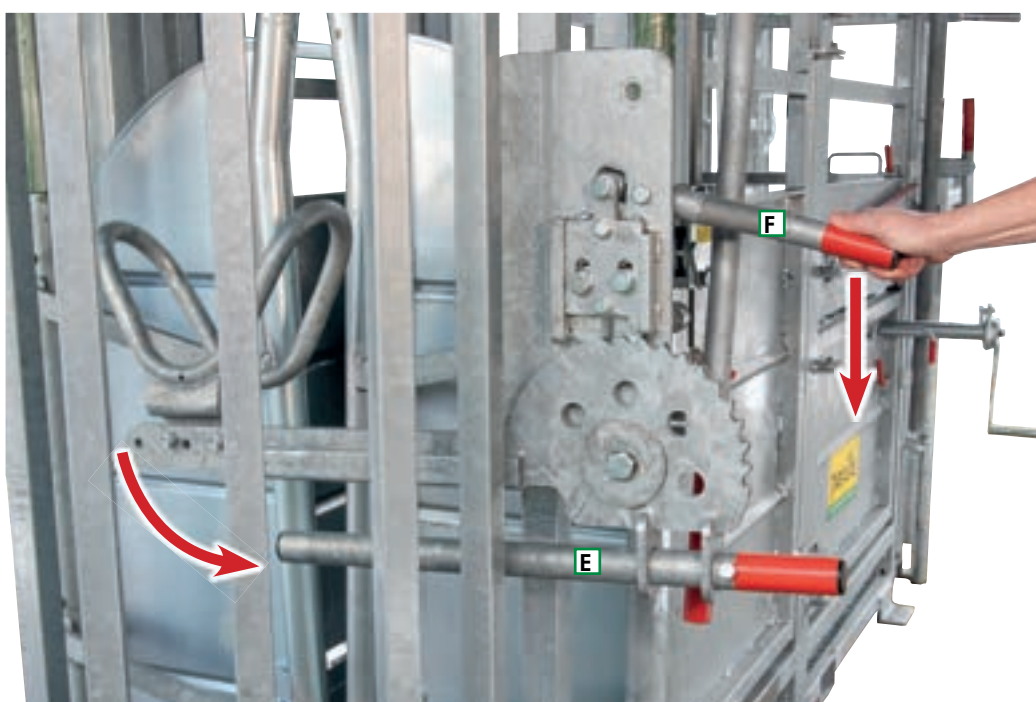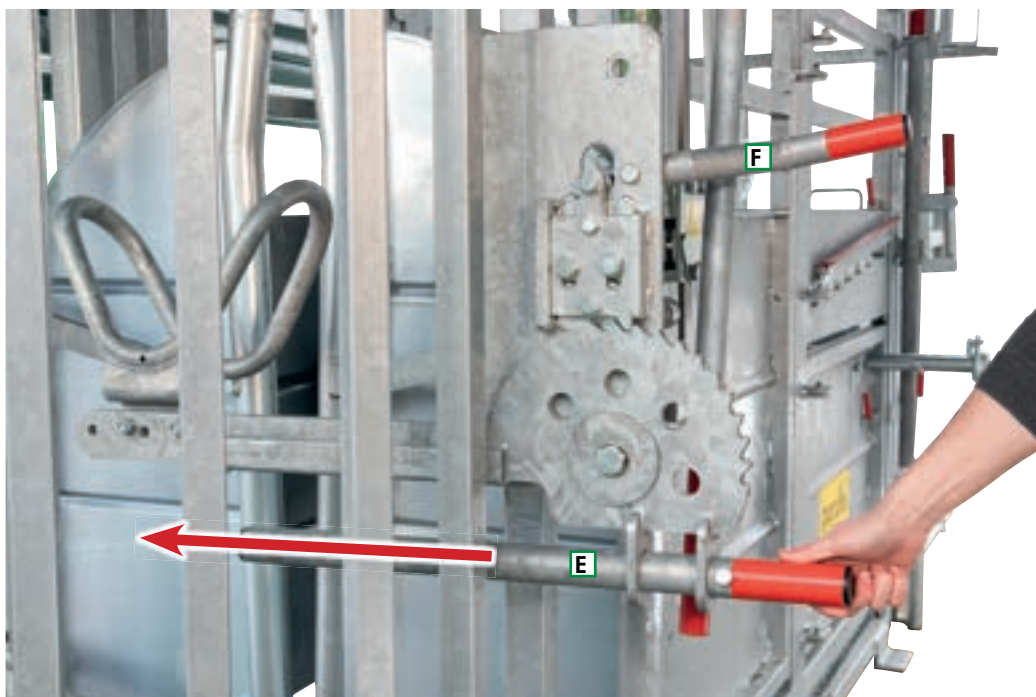⚠ Before doing so, make sure to push the operating lever **E** back in!

### 8. Installation Weighing System 5 (optional)

**A** Lift the cattle crush safely and install the load bars onto the integrated mounting plates of the crush. Do not tighten the screws yet!

Store the load bar cables safely and place the crush on an even surface.

When all 4 feet of the load bars have full ground contact, the screws can be tightened.

Use the enclosed anchor bolts for stationary use.

**B** Attach the indicator unit with the enclosed mounting material to the crush and secure the cables so that they cannot be damaged.

# PATURA Cattle Crush A6000

10 advantages that will convince you

**1** Baulk gate for easy and safe capture

**2** Infinite headbail to restrain all animal sizes (80 - 12 cm)

**3** Optimal traction due to steel mesh floor

**4** 3 side gates on both sides for easy access to all body parts; Simple transport via pallet fork

**5** Hoof Care Equipment with recoilless winches

**6** Rear sliding gate for quick and safe closing

**7** Animal restraint from the rear with rump bar

**8** Hinged doors in rear sliding gate for hoof care

**9** Hooks for belly strap can be adjusted as required

**10** Integrated mounting plates for load bars HDST

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